# Harnröhren

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(Modell Nürnberg)

---

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Réf. : MED MHR

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Largeur : 2 cm

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## Urethral Closure Band (Nuremberg model)

---

### Indications:

The urethral closure band is a circular compression cuff which is easily applied to the penis. A soft compression pad prevents medicines injected into the urethra (liquids, gels, contrast means, etc.) from running out and similarly prevents the inadvertent loss of urine in mild to moderately severe urinary incontinence.

### Advantages:

- the soft, padded, pleasant to wear cuff causes less pressure pain than comparable penis clamps
- the Velcro fastener simplifies adjustment
- the compression effect is easily adjusted by the patient
- light to wear and small in size the device provides good wearing comfort when running, sitting, walking
- Disposable

### Application:

Open the cuff to approximately the diameter of the penis. Align the pressure pad when slipping it over the penis to the lower surface of the penis and the urethra. Tighten the closure band and close using the Velcro fastener. The urethra

should be compressed, but not uncomfortably so or painful. Please check for about 15 min after fitting to ensure that the glans has a "normal" colour and is not dark red or blue. In the case of bluish discoloration, the pressure of the seal should be loosened slightly. In order to pass urine, the small band is opened a little, left on the penis and held with one hand and pushed in the direction of the body. Afterwards, the small band can be pushed in the direction of the glans again and closed.

### Material:

frontside: polyamid (100 %)  
backside: cotton (100 %)  
foam rubber: polyester (100 %)

### Contraindications:

If you are allergic to cotton, polyamid or polyester, please contact your healthcare provider before use of this device.

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Kessel Marketing & Vertrieb GmbH