Repotento[®]

Erection Tape

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Repotento® *Erection Tape*

Method of action

Repotento® *Erection Tape* is a self-adhesive, elastic tape. The start of the tape is stronger and allows the penis to be artificially stiffened from the outside. The glans (head of the penis) remains uncovered for sensitivity. Using **Repotento® *Erection Tape*** makes normal, satisfactory sexual intercourse possible.

Application

To apply the **Repotento® *Erection Tape***, we recommend that you sit on the edge of a chair. You may need to remove some pubic hair. After you have removed the protective foil, unwind about 10 cm of tape from the roll and keep this to hand. Now pull back your foreskin until the glans is exposed.

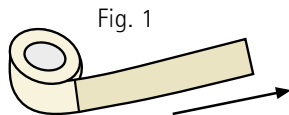
Fig. 1

Unwind 10 cm of tape

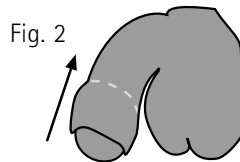
Fig. 2

Pull back the foreskin

Using your fingers, gently pull the glans away from your body. Start the first wind from bottom of the penis. Use light pressure.

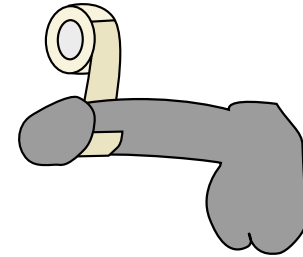
Fig. 3

Now using your other hand, wind the tape around the penis. Start with two layers around the corona glandis [base of the glans], winding in the direction of the root of the penis (towards your body).

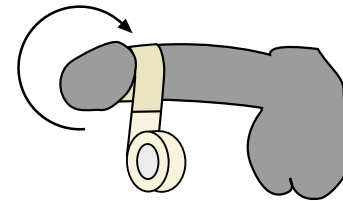
Fig. 4

Wind the rest of the tape around your the penis, covering it completely and evenly. The glans must not be covered. The tape is self-adhesive and winding it around itself is enough to make the end of the tape stick.

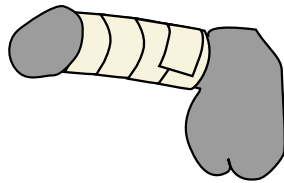
Fig. 5

We recommend that you cover your penis with a condom, as otherwise the tape might feel too rough. This will also help to fix the end of the tape better. The condom is moistened with silicon oil so that it can glide better. Men who have had prostate operations may experience a urine leak when they ejaculate.

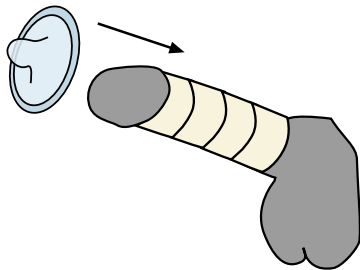
Fig. 6

This will be contained by the condom. The artificial stiffness achieved in this way allows for up to 30 minutes of sexual activity. The tape must be removed afterwards.

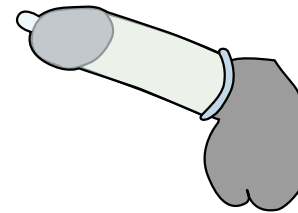
Fig. 7

Further Information

Be careful not to wind the tape too loosely or too tightly around your penis. **Repotento® Erection Tape** is intended for single use only. If pubic hairs cause any problems, you can easily remove them from the shaft of the penis using a razor.

Material

Cotton, polyamid, latex rubber, Please note, that this product is made by natural latex rubber.

Dimensions

Tape length (unstretched) 165 cm / width: 4 cm / thickness: 1 mm

Contraindications

Redness of the skin can occur with latex allergies. Should this happen, stop using the product at once. If necessary, consult your doctor. Applying **Repotento® Erection Tape** requires some dexterity. This will become easier after some practice.

Risks

If the tape is wound too tightly less blood circulation of the glands might change the color of the glands penis to blue. In this case, reduce the pressure of the tape.

Further information: **Repotento® Erection Tape** can be used for up to 30 minutes. It is intended for single use only.

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Distler Medizintechnik GmbH